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Activity Meter

*with Bluetooth and data
management in a dedicated app*

Operating Manual

AM 1400
adeVital
Activity

We reserve the right to make technical modifications in the course of further development.

Operating Manual_AM1400_EN_151106_REV004

THANK YOU,

for choosing to purchase this activity meter from the adeVital product family. All of the products of the innovative **adeVital range – activity meter, bathroom and body analyser scales, and blood pressure monitor** – are equipped with a Bluetooth function for transferring all data to the adeVital^{plus} app. The adeVital^{plus} app is available for free at the App Store and the Google Play Store. The clearly arranged user interface of the adeVital^{plus} app enables simple and clear progress and results monitoring. Health values are therefore available at any time and are always up-to-date.

SAFETY AND USAGE INFORMATION

SAFETY INFORMATION

Warning signs and symbols are important for the correct and safe use of this product, and for protecting you and third parties from injury. The warning signs and symbols that you will find on the labels and in this operating manual have the following meaning:

Symbol for: “Read the user manual and follow the safety and warning information”

ATTENTION!

The Activity meter AM 1400 from ADE offers you a simple way to monitor your health.

The Activity Meter is equipped with a Bluetooth 4.0 data transmission function. In order to fulfil the intended functions, electromagnetic energy may be released. RF communication devices located in the vicinity can affect the performance of the activity meter. Portable and mobile RF communications equipment can affect electrical devices. Electrical devices are subject to particular precautions with regard to EMC and must be operated in accordance with this operating manual. Be aware that the misuse of electrical devices can lead to electric shocks, burns, fire, and other dangers.

USAGE INFORMATION

With the **ACTIVITY METER AM 1400** you can measure the number of steps you take, the distance covered, and the calories burned. In addition, you can monitor your sleeping times in the *adeVital^{plus}* app.

A prerequisite for using all the performance parameters of this device is the use of the free *adeVital^{plus}* app on a mobile device. Compatible devices are the iPhone4S, iPhone5, iPhone5S, iPhone6, iPhone6 plus, iPod touch (5th generation), iPad3, iPad Air, iPad mini and later versions, as well as Android devices from version 3.0 onwards. Bluetooth 4.0 compatibility is required.

The user can input personal settings in the app and transfer these via Bluetooth to the **ADE Activity Meter AM 1400**. The data of the **ADE Activity Meter AM 1400** is transferred via Bluetooth only to a device that supports Bluetooth 4.0 and on which the *adeVital^{plus}* app is installed.

Important features:

- a. Permanent record of steps/activity
 - b. Display of daily activity (hours/minutes) with 24-h display
 - c. Display of calorie consumption
 - d. Display of data on the display and in the *adeVital^{plus}* app
 - e. Sleep monitoring (data is displayed only in the *adeVital^{plus}* app)
 - f. Faster and safer data exchange using the *adeVital-plus* app
 - g. Coupling of the *adeVital^{plus}* app with the Apple Health app
 - h. OLED Display
 - i. Supports Bluetooth 4.0 technology
 - j. Measured data is stored on the activity meter for 7 days and is recorded every 5 minutes during changes
 - k. Number of steps, distance covered (miles/km), calories consumed (Kcal), and current time are displayed
 - l. The current consumption is below 1 mA.
- The firm Waagen-Schmitt accepts no liability for damages or losses caused by the activity meter, or for the claims of third parties. This product is intended exclusively for private use.

However, please keep in mind that persons with portable or implanted medical electronic devices such as pacemakers may use the activity meter only after consultation with their treating doctor.

- The ascertained data is made available in the linked **ADEVITAL^{PLUS} APP** and is displayed there in diagrams and remains retrievable for up to one year.

AVOID STORING THE DEVICE IN THE FOLLOWING LOCATIONS:

- In the vicinity of water
- Where the device is exposed to extreme temperatures, humidity, moisture, direct sunlight, dust, or salty air
- Where there is a risk of bumping or falling
- In the vicinity of chemicals or corrosive gases
- Within reach of infants or children

PACKAGING CONTENTS

- Activity meter AM 1400 with permanently installed 3.7 V 70 mAh rechargeable lithium polymer battery
- Armband for various arm circumferences, with a total length of 135 – 235 mm.
- Operating Manual

START-UP

ATTENTION!

Remove the activity meter and the operating manual from the packaging and read through the operating manual carefully prior to first use.


Prior to first use, the *ADE ACTIVITY METER AM 1400* must be charged via a USB connection on a computer (PC, tablet, notebook). In doing so, the underside of the activity meter must be pointed at the USB symbol on the computer port.

To start the Activity Meter, press the button on the side for around 5 to 10 seconds, so that the device enters start mode.

COMPONENTS OF THE ACTIVITY METER

- Body with OLED display
- Operating button
- USB connector
- Armband with clasp for adjustment of the armband
- Clasp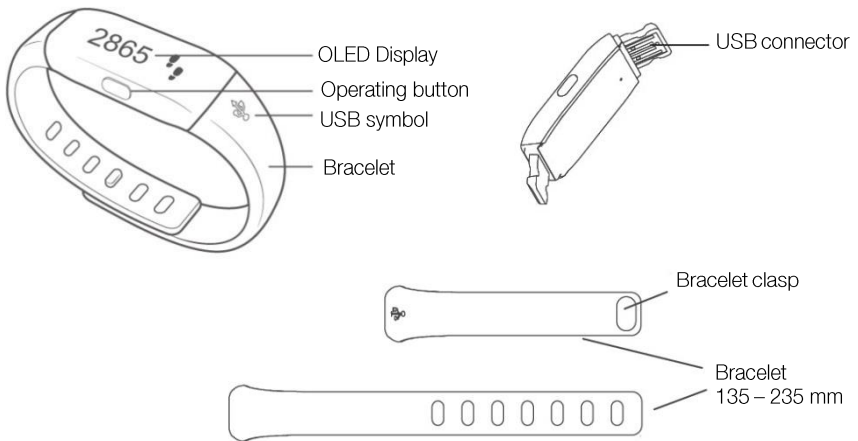

OLED DISPLAY


SYMBOL	DESCRIPTION
	steps
	Current time
	Calories consumed (kcal)
	Distance covered

CHARGING THE BATTERY


When the following symbol appears on the OLED display, it is to remind you to charge the battery.

In order to charge the battery, connect the USB connector of the Activity Meter to the USB port of a computer (PC, Tablet, Notebook) or a USB charging device. Please pay attention to the contacts of the USB connector.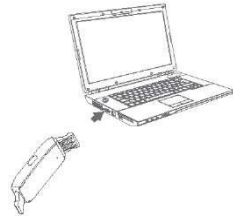

When the battery of the device is being charged, the following image appears on the OLED display. The energy symbol blinks.

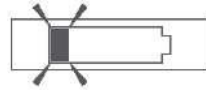
When the battery of the device is fully charged, the following image appears on the OLED display.

PUTTING ON THE BRACELET

1. Attach the two individual parts of the bracelet to the Activity Meter (please make sure that the bracelet part with the USB symbol is connected to the USB connector)
2. Close the armband using the bracelet clasp so that it is closed firmly on your arm but is nevertheless comfortable to wear.

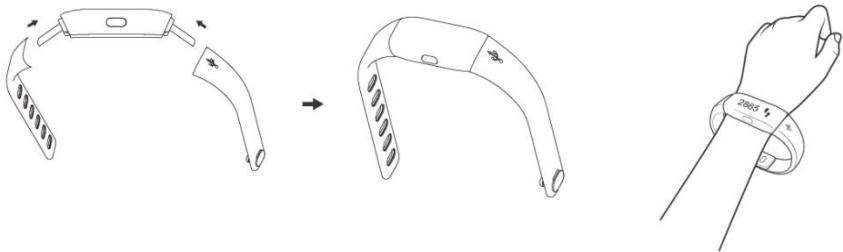Wear the Activity Meter as you would a wristwatch. Bear in mind that the side of the bracelet with the USB symbol must point towards the body.

You can now begin monitoring.

ATTENTION!

If you are wearing the Activity Meter on your left wrist, the button on the Activity Meter must point towards your arm. If you are wearing the Activity Meter on your right wrist, the button on the Activity Meter must point towards your fingertips.

INSTALLING THE ADEVITAL PLUS APP AND CONNECTING

Using Bluetooth 4.0 technology, mobile devices with a Bluetooth function can receive your personal data.

The adeVital^{plus} app is available for free at the App Store (for iPad users, search under using an iPhone) and at the Google Play Store. There you can search for the “adeVital plus” app and install it on your mobile device.

MONITORING FOR ACTIVITY METER

- Display of daily data for all ascertained values
- Steps (steps), calorie consumption (kcal), distance (miles/km), duration of activity (hrs/min), sleep duration (hours/minutes)
- Diagram for displaying average values
- Generation of progress curves for various intervals
- Presentation as graph
- Simple data transmission via integrated email function, Facebook, or Twitter

Simply install the specially developed adeVital^{plus} app and connect your Activity Meter using your mobile or portable devices. You can then make use of the comprehensive software in the adeVital^{plus} app.

The app, for connecting the Activity Meter, is available for iPhone4S, iPhone5, iPhone5S, iPhone6, iPhone6 plus, iPod touch (5th generation), iPad3, iPad Air, iPad mini and later versions as well as devices for Android from version 3.0 onwards. Bluetooth 4.0 compatibility is required.

COUPLING THE ACTIVITY METER AND THE ADEVITAL PLUS APP

Here, the coupling with an iPhone is described and depicted. Images/displays may differ. Function and concepts similar with android.

– Activate Bluetooth, open the installed adeVital^{plus} app and input your personal data in all of the given fields. You will find the “Administration” menu under the profile icon.

– Under “Devices” in the “Administration” menu, select the large “+ ADD DEVICE” symbol. At the same time, switch the AM 1400 on by tapping once quickly on the operating button and then press and hold the button again for approx. 10 seconds until the symbols  and  appear.

– The symbols blink alternately on the display and indicate that “linking” is in progress.


– As soon as the device has been found, it is displayed in the adeVital^{plus} app. Now tap on the AM 1400 logo.

– If the connection is successful, **[Success!]** appears on the LCD display.

– If the connection fails, **[Fail!]** appears on the LCD display.

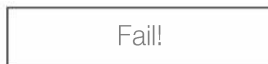COUPLING OF THE ADEVITAL PLUS APP WITH APPLE HEALTH

adevital plus

Health

If you are using the Apple Health app, you can couple this app with the adevital^{plus} app and then analyse your personal values in the Apple Health App, too.
Not available for iPad, not available in the AppStore.

To couple the two apps, open the 'Administration' menu in the adevital^{plus} app and proceed to 'Settings' → 'Health-Kit'. In the "Fitness" category, activate the data that you would also like to analyse in the Apple Health app. Then open the Apple Health App and select the "adevital plus" app in the 'Source' menu. In the 'Fitness' category, activate the data corresponding to your selection in the adevital^{plus} app.

USE

After you have successfully connected your Activity Meter with your mobile device and the adevital^{plus} app is active, you can immediately start taking measurements. Recording takes place automatically.

Start to walk
(fewer than 10 steps)

10 steps

More than 10 steps
(the number increases)

The measurement data will be automatically transferred to your device, which supports Bluetooth 4.0, once the connection has been made.

SETTING THE SLEEP MONITORING

1. Setting Option

Press the button on the Activity Meter for approximately 5 seconds when you are going to sleep, until a small, blinking moon appears on the right of the display. Sleep mode is now active. When you get up, press the button again for approx. 5 seconds until the moon no longer appears. The Activity Meter is now in normal day mode.

2. Setting Option

In the “Sleep” area in the adeVital^{plus} app, tap on the “Clock/Alarm Symbol” at the upper right and activate “Set Sleep Mode according to Time”. Set the start and end time of your desired sleep period and activate or deactivate the “Alarm” setting. Then click the side push button on the ADE Activity Meter AM 1400. The settings will be transmitted to the Activity Meter. Sleep mode will now be automatically activated and deactivated according to your set times.

SETTING THE ALARM

In the Activity area in the adeVital^{plus} app, tap on the “Clock/Alarm Symbol” and tap on one of the three possible alarms. Set the desired alarm time and days when you would like to be woken by the alarm. You can then activate or deactivate the three alarms in the overview area of the alarm settings. As soon as your settings have been adjusted, press briefly on the side push button on the Activity Meter in order to transmit the alarm settings to the monitor (Bluetooth must be switched on). The successful transmission will be indicated to you in the upper area of the adeVital^{plus} app with “Alarm is available”.

DATA DISPLAY

ATTENTION!

In the vicinity of devices that bear the  symbol, interference is possible. The Activity Meter can interfere with electrical devices in the surroundings.

For data transmission, this product must be paired with a Bluetooth device at 2.4 GHz.

How can potential interference be reduced?

1. The distance between the Activity Meter and the Bluetooth device should be as small as possible, between 1 m and 10 m. There should be no obstacles to impair the Bluetooth connection between the Activity Meter and the Bluetooth device.
2. To avoid disturbances, you should keep other electronic devices (in particular those with Bluetooth transmission/transmitters) at least 1 m away from the Activity Meter.

If the OLED display is switched off, press the button on the Activity Meter. If enough current is available from the battery, the OLED display shows the indicators that were shown before it switched off.

Example:

Before the Activity Meter switched off, the OLED display was showing the calories you consumed. If you now press the operating button, the OLED display shows the calories you consumed.

If there is not enough current available from the battery, the OLED display is off. If you press the operating button on the Activity Meter, the charge status of the battery will first be shown on the OLED display.


Press the operating button again. The OLED display then shows the distance you covered.

Press the operating button again. The OLED display then shows your number of steps.

Press the operating button again. The OLED display then shows the time.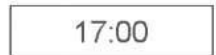

If no activity takes place, the OLED display switches off after a few seconds.

ERROR MESSAGES AND THEIR CORRECTION

Failure	Cause	Solution
	Connection process has failed	Please check the following: <ul style="list-style-type: none"> – Bluetooth is ON (activated) – App is ON (activated/open). The distance between the devices lies within the maximum Bluetooth transmission range (10 m)

Abnormal measurement results: Large difference between two of the last measurements Abnormal measurement results: Large difference between two of the last measurements	The Bluetooth symbol of the AM 1400 is not pointing to your body	Attach the device correctly to your arm, avoid irregular shaking of the Activity Meter
	Fewer than 10 steps	Take more than 10 steps
	Sliding over the floor or slippers on your feet	Walk normally
No OLED display when the device is switched on	Battery is low	Charge battery
The device switches off	Battery is low	Charge battery.
Data transmission failed	Bluetooth is OFF (deactivated)	Switch Bluetooth on via "Setting >> General >> Bluetooth".
Data transmission failed	The app is OFF (deactivated)	Tap the symbol of your app in order to start it
	Outside the maximum Bluetooth range	Place your Bluetooth 4.0 device closer to the Activity Meter

MAINTENANCE

Please read the following directions for maintenance of the device:

Use a soft, dry cloth to wipe dust off the device.

Use a soft, damp cloth dipped in water and squeezed out, in order to remove grime from the device. Then use a soft, dry cloth to wipe the device dry.

Do not wash the device with water, and do not submerge it in water.

Do not use any cleaning agents, abrasive or other chemicals to remove grime from the device. This can lead to colour changes or malfunctions.

The device must not be dismantled. If you encounter problems, please contact Waagen-Schmitt GmbH.

(For contact information, see Warranty and Guarantee)

WARRANTY AND GUARANTEE

Waagen-Schmitt GmbH guarantees the cost-free rectification of defects due to material or fabrication errors through repair or exchange for 3 years from the purchase date. Please get your warranty section filled out and stamped by the retailer when you make your purchase. In the event of a guarantee claim, please return the Activity Meter to your dealer along with the warranty section specifying the reason for your complaint.

Manufactured for:

Waagen-Schmitt GmbH

Hammer Steindamm 27 – 29, 22089 Hamburg

Made in P. R. C.

DISPOSAL

Legal duty to inform about battery disposal:

Batteries do not belong in household waste. As a consumer you are legally obligated to return batteries. You can hand in your old batteries at the public collection point in your community or anywhere where batteries of the same kind are sold.

Note:

These symbols can be found on batteries:

Li = Battery contains lithium

Al = Battery contains alkali

Mn = Battery contains manganese

CR (Li); AA (Al, Mn); AAA (Al, Mn)

Safety information for lithium batteries (CR2032/CR2430):

1. Please never open the lithium batteries, nor throw them into fire or subject them to impacts, since it is possible that poisonous fumes could escape or there could be a risk of explosion.
2. Never grab leaking power cells with your bare hands.
3. It is imperative that contaminated eyes or hands be rinsed thoroughly with water. If skin or eyes become irritated, medical assistance should be sought immediately.
4. Only dispose of completely discharged cells or cells protected against polarity reversal, in accordance with local disposal regulations.
5. Do not subject the power cells to direct sunlight or large amounts of heat, otherwise there is a risk of overheating.
6. Maintain a storage temperature of <math><40^{\circ}\text{C}</math>.

Disposal of used electrical and electronic devices:

This symbol on the product or its packaging indicates that the product is not to be treated like normal household waste, but must be handed over at a collection point for the recycling of electrical and electronic devices. Further information can be obtained in your community, at communal disposal services or the business where you bought the product.

DECLARATION OF CE CONFORMITY

Declaration of conformity by the manufacturer

This device has been manufactured in accordance with the harmonised European standards. It complies with the provisions of the following EC directives:

- EMC Directive 2004/108/EC regarding electromagnetic compatibility
- RoHS Directive 2011/65/EC

in the currently valid versions.

The declaration loses its validity if a modification has been made to the device without our prior approval,

Hamburg, November 2015

WAAGEN-SCHMITT GmbH

Hammer Steindamm 27 – 29

D-22089 Hamburg

TECHNICAL DATA

Product name:	Activity Meter AM 1400 adeVital Activity with app
Dimensions:	135 – 235 x 20 x 10 mm
Net weight:	approx. 21 g (including battery)
Display:	Blue OLED display, Display surface: 22.4 x 5.6 mm
Steps:	0-120,000 steps, Resolution 1 step
Calories:	0-12,000.0 kcal, Resolution 0.1 kcal
Exercise time:	0-23 hours and 59 minutes, Resolution 1 minute
Battery:	3.7 V/70 mAh rechargeable lithium polymer battery
Charge state indicator, battery:	1-6 segments, Resolution 1 segment
Battery life:	≥ 7 days when fully charged and with 8 hours of usage per day
Accuracy:	0-50 kg: ±0.3 kg; 50-100 kg: ±0.4 kg; 100-150 kg: ±0.5 kg; 150-180 kg: ±0.7 kg
Operating conditions:	Temperature: 0 to 40°C Humidity: ≤90% rel. humidity Air pressure: 80 kPa ~ 106kPa
Storage conditions:	Temperature: -20 to 60 °C Humidity: 10% to 90% rel. humidity Air pressure: 50 kPa ~ 106kPa
IP Protection:	IP65

Warranty Section – 3 Years on the Activity Meter

- Sender

- Reason for complaint

- Date of Purchase _____

Please send this section incl. proof of purchase back to your specialist retailer.

Operating Manual_AM1400_EN_151106_REV004.docx

Collect, manage, and analyse health data: Simple and clear using the new adeVital product series with Bluetooth functionality and the intelligent adeVital^{plus} app.

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